

Contents

1.Product Description	2
2. Important Information Before Using Black Origin™	2
3. Interaction with Other Medicines or Treatments	3
4. Post-Treatment Care	3
5. Are There Any Side Effects?	3
6. Benefits	4
7. What if I experience a problem?	4

1. Product Description

What is Black Origin™?

BLACK ORIGIN™ is a hyaluronic acid-based injectable gel, a naturally occurring substance in the body. It aims to improve skin texture, reduce wrinkles, and enhance overall skin quality. BLACK ORIGIN™ is administered using an ultrafine needle and is suitable for treating the lower face, mid-face, neck, and other areas of the body.

2. Important Information Before Using Black Origin™

Warnings:

- BLACK ORIGIN™ should only be administered by a qualified medical practitioner or a nurse injector under medical supervision.
- An initial cycle of two treatment sessions, spaced 30 days apart, is typically recommended. Your healthcare provider will determine the best schedule for you.

Do Not Use *Black Origin™* If:

- You are allergic to hyaluronic acid or any other ingredients listed at the end of this leaflet.
- You have severe allergies, such as a history of anaphylaxis or multiple severe allergies.
- You have a bleeding disorder.
- There is an active inflammatory condition (e.g., skin eruptions like cysts, pimples, rashes, or hives) or an infection in the treatment area.

Precautions:

- *Black Origin™* should not be used alongside treatments like laser resurfacing or medium-deep chemical peels at the same time.
- Consult your healthcare provider if you are pregnant, planning to become pregnant, or breastfeeding, as safety has not been established for these situations.

3. Interaction with Other Medicines or Treatments

Inform your doctor or injector if you are taking any other medications, including vitamins, supplements, or topical skincare products. No known interactions exist between *Black Origin™* and other drugs.

4. Post-Treatment Care

Things to Avoid:

- Physical exertion, head-first movements, and sports activities on the day of treatment and the following day.
- Saunas, steam baths, extreme temperatures, or UV radiation (e.g., tanning beds) for two weeks post-treatment.
- Dental procedures for two weeks after treatment.
- Avoid other treatments such as filler injections in the same area, microdermabrasion, chemical peels, and IPL for two weeks post-treatment.
- Avoid fractional resurfacing for four weeks after treatment.

To maintain optimal results, protect treated areas from sun exposure by using sunscreen.

5. Are There Any Side Effects?

As with any medical product, *Black Origin™* may cause side effects. These are typically minor and temporary, but some may require medical attention.

Possible Side Effects:

- Pain, redness, swelling, or itching at the injection site, which can be alleviated by applying ice. These symptoms usually resolve within 10 days.
- Hardening or nodules may develop at the injection site.
- Rare side effects include discoloration, necrosis (tissue death), abscesses, granulomas, or hypersensitivity reactions.

Contact your healthcare provider if you experience any of these side effects or if they cause concern.

6. Benefits

What will Black Origin™ do for me?

Black Origin™ offers several benefits for individuals seeking to improve their skin quality:

- **Enhanced Skin Hydration:** The hyaluronic acid in *Black Origin™* helps to deeply hydrate the skin, resulting in a more youthful and glowing appearance.
- **Improvement in Skin Texture and Elasticity:** By boosting collagen production, *Black Origin™* can enhance skin texture, making it smoother and more elastic.
- **Reduction of Fine Lines and Wrinkles:** *Black Origin™* effectively reduces the appearance of fine lines and wrinkles, leading to firmer, younger-looking skin.
- **Natural-Looking Results:** *Black Origin™* uses a naturally occurring substance in the body, providing subtle and natural-looking results without drastically altering facial features.
- **Safe and Minimally Invasive:** As a non-surgical treatment, *Black Origin™* is minimally invasive and involves little to no downtime, allowing patients to return to their daily activities shortly after treatment.

Consult your healthcare provider to determine how *Black Origin™* can best meet your aesthetic needs immediately.

7. What if I experience a problem?

- If you believe that you have experienced a serious problem related to *Pol Tech* products, you should contact your doctor immediately. If you have any questions regarding *Pol Tech* products, you can personally reach out to our customer support team. Our representatives are available to assist you between 8:00 a.m. and 5:00 p.m. (local time), Saturday through Thursday.
- Please contact us at: 021-284208